

Le Perch Outdoors

OWNERS MANUAL



model TCR210

Trackchair® Canopy



BE SURE THE L HOOKS ATOP THE SUPPORT TUBE ARE FACING REARWARD TO PREVENT THE CANOPY FROM RELEASING AND COLLAPSING UPON THE USER.



USE IN WINDY CONDITIONS OR HIGHER SPEED CHAIR TRAVEL COULD RESULT IN COLLAPSE OF THE CANOPY CAUSING INJURY TO THE USER.

PARTS/SPECIFICATIONS

1 each: 42" x 48" trapezoidal shaped waterproof Ottertext™ ripstop canvas with an incorporated fully shock corded 42" x 38" frame of 3/8" diameter fiberglass tubing

1 each: 3/4" square x 36" powder coated flat black support tube with 12° bend and top mount 6-32 L hooks

1 each: 1" square x 4 1/4" powder coated flat black T grip

1 each: 5/8" hook & loop bundle strap

1 each: storage pouch with carry strap

SETUP AND USE

After unpacking all pieces, slide the T grip over the 3/4" square x 36" support tube with the T grip facing opposite the knee of the support tube and temporarily lock in place approximately 8" from the top by tightening the thumb screw. (see Figure 1) Insert the support tube into the 1" square receiver on the Trackchair® seat frame with the knee of the support tube facing rearward. Since the length of the Trackchair® receivers vary in length; be sure at least 1 1/2" of the support tube is inserted into the receiver before locking in position with the Trackchair® receiver thumb screw.

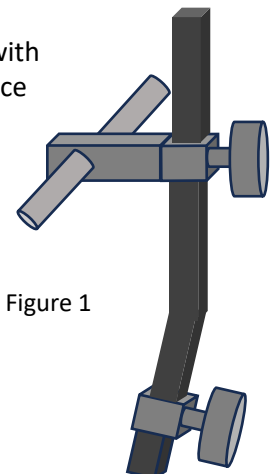


Figure 1

Unfold the canopy with incorporated shock corded tubular frame and engage all fiberglass frame segments into their adjacent connectors. Slip the frame front portion into the pocket of the canopy front and close using the attached snaps. Gripping the rear of the canopy canvas pull the canopy canvas taught over the frame. From the front of the Trackchair® with the canopy top facing up, insert the rear frame segment ends into the couplings of the T grip. Turn the L hooks in the top of the support tube to point rearward. Holding the canopy frame, raise it above horizontal and slide the two canopy eyelets over the two L hooks at the top of the support tube. Slowly lower the canopy/frame towards the horizontal position causing the excess canvas between frame sides to form a sloped surface. Adjust the position of the T grip by loosening the thumb screw and raising or lowering it until the canvas is taught and the frame is approximately horizontal. Lock the T grip position by tightening the thumb screw.

Warranty

Le Perch Outdoors, LLC warrants as limited herein to the original purchaser of retail that each new Trackchair® Canopy, shall be free of defects in material and workmanship for a period of one (1) year from date of original purchase.

In the event of malfunction or failure of your Le Perch Outdoors Trackchair® Canopy, simply send the complete unit, postage paid along with PROOF OF PURCHASE, within the warranty period of one (1) year, to: Le Perch Outdoors, LLC, 8406 Lylwood Ct., Chesterfield, VA 23838. Le Perch Outdoors, LLC reserves the right to inspect the claimed defective part(s) to determine if the defect or malfunction complaint is covered by this warranty. Le Perch Outdoors, LLC shall within sixty (60) days of receipt of the product, at its option, repair and/or replace the defective part(s) free of charge. This warranty shall only cover defects arising from normal usage. Le Perch Outdoors, LLC assumes no responsibility whatsoever if the Canopy should fail during the warranty period by reason of

- Misuse, negligence, physical damage or accidents.
- Repair by an unauthorized party during the warranty period.

Le Perch Outdoors, LLC makes no further warranties or representations, expressed or implied except those contained herein. Le Perch Outdoors, LLC shall in no event be liable for direct, indirect, special or consequential damages

Le Perch Outdoors, Chesterfield, Virginia 23838 (804) 930-4415 www.leperchoutdoors.com